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EN

COMPOSITION Trehalose dihydrate (1,9%), Vegetable waters (Chamomile, Cornflower flower, Sweet clover, Euphrasia): 10,9%, Sodium chloride. Buffered solution.

INDICATIONS Ideal in case of: • Eye dryness, irritated, tired eyes • Tingling, burning or foreign body sensations, eye blink difficulties, blurred vision, itching • Non-traumatic / non-pathological alterations of the tear film • Provides quick relief in case of ocular stress due to: - excessive exposure to sunlight, wind, dust, pollen, chlorine, air conditioning, dry or polluted air - prolonged computer use and indicated for prolonged use of contact lenses.

INSTRUCTIONS FOR USE 1/ Remove the ring around the cap • 2/ Remove the cap (if necessary prime the flow of the vial by pressing it over your hand until you get the first drop) • 3/ Turn the bottle over and position it over the eye • 4/ Open your eyes and press the bottle to place 1 to 2 drops in the corner of each eye, up to 4 times a day • 5/ Do not touch the eye with the dropper • 6/ Close the cap carefully and put it back in its original case to keep it away from light.

WARNINGS • Suitable for adults and children over 15 years • Do not use if the bottle is damaged • Do not ingest • Keep out of the sight and reach of children • Do not use with another eye solution • In case of eye irritation, persistence of symptoms or any adverse effects, stop using the product and contact a doctor • Do not use the product continuously for more than 28 consecutive days.

CONTRAINDICATION • Do not use in case of specific hypersensitivity towards any of the constituents.

PRESERVATION • Use within 30 days after opening • Store in a dry place away from light • Do not exceed the cut-off date on the packaging.

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