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👁️ Periocular lotion. Sensitive eyes and eyelids. Cleanses - Relieves.

Composition: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

Indications: Hydravision Periocular lotion is a medical device for the hygiene of the eyelids and the periorcular area of adults and children (from 3 years old) indicated in case of: Inflammations and/or infections - Blepharitis and blepharocconjunctivitis - Allergic inflammations - Pre- and/or post-operative phases of surgery or laser treatment - Suitable for irritated skin with eczema or dermatitis.

Instructions for use: Wash your hands thoroughly - Dampen a cotton disk or gauze with the solution and wipe, without rubbing, both the eyelid area, applying it gently so as to ensure effective removal of the ocular secretions and any other deposits present (smog, pollen, dust, makeup, etc.) - Recommended frequency: once a day.

Warnings: Store in a cool, dry place - Keep out of the reach of children - For external use - Discontinue treatment in case of severe irritation or hypersensitivity to the product - Do not use the product after its expiry date - Do not use the product in case of known hypersensitivity to any of the ingredients - Do not swallow - Do not apply to injured skin - After first opening, use the product within three months. Any serious incident related to the medical device that we supply must be reported to the manufacturer and the competent authority of the Member state where it occurred.

👁️ Perioculaire lotion. Gevoelige ogen en oogleden. Reinigt - Verlicht.

Samenstelling: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Zusammensetzung: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Composizione: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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成分: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Composición: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Composição: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Σύνθεση: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Bileşim: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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المكونات: Chamomilla recutita extract, Euphrasia officinalis extract, Sodium chloride, Sodium hyaluronate, Aloe barbadensis leaf juice powder, Vitamin B3, Aqua, Propylene glycol, Cocoyl wheat amino acids, Disodium phosphate, Ammonium glycyrrhizate, Ethylhexylglycerin, Phenethyl Alcohol, Sodium phosphate, PEG-40 hydrogenated castor oil, PEG-40 castor oil, Polysorbate 20.

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Promeia Italia Srl