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EN MOISTURISING EYE DROPS BLUE FORMULA. RED & TIRED EYES. EYEBRIGHT - RELIEF.

COMPOSITION: Trehalose dihydrate (1,9%), Floral waters (Chamomile, Cornflower flower, Sweet clover, Euphrasia): 10,9%, Methylene Blue, Sodium chloride, buffered solution.

INDICATIONS: Ideal in case of: • Eye dryness • Red, irritated, tired eyes • Tingling, burning or foreign body sensations, eye blink difficulties, blurred vision, itching • Non-traumatic / non-pathological alterations of the tear film • Provides quick relief in case of ocular stress due to: - excessive

Hydrobleu
Hydroblue

exposure to sunlight, wind, dust, pollen, chlorine, air conditioning, dry or polluted air - prolonged computer use.

INSTRUCTIONS FOR USE: 1/ Remove the ring around the cap • 2/ Remove the cap (if necessary prime the flow of the vial by pressing it over your hand until you get the first drop) • 3/ Turn the bottle over and position it over the eye • 4/ Open your eyes and press the bottle to place 1 to 2 drops in the corner of each eye, up to 4 times a day • 5/ Do not touch the eye with the dropper • 6/ Close the cap carefully and put it back in its original case to keep it away from light. • It is advisable to protect clothing before use.

WARNINGS: • Not recommended when wearing contact lenses • Suitable for adults and children over 15 years • The product is sold sterile. Do not use if damaged or already open • Do not ingest • Keep out of the sight and reach of children • Do not use with another eye solution • In case of eye irritation, persistence of symptoms or any adverse effects, stop using the product and contact a doctor • Do not use the product continuously for more than 28 consecutive days.

CONTRAINDICATION: Do not use in case of specific hypersensitivity towards any of the constituents.

PRESERVATION: • To be used within 3 months after opening • Store in a dry place away from light, in temperatures of between 15°C and 30°C • Do not exceed the cut-off date on the packaging.

These medical devices are regulated health products which carry CE0051 marking, in accordance with this regulation. Any serious incident that has occurred in relation to the device should be reported to the manufacturer at the address: client@biosynex.com and the competent authority of the Member State in which the user is established.

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QARAD UK Ltd