Spray Oculaire Œil et paupière  
Yeux rouges et irrités  
Apaise - Hydrate - Rafraîchit  
Ocular spray eye and eyelid  
Red and irritated eyes  
Soothes - Moisturises - Refreshes

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## EN Ocular spray Eye and eyelid – Red and irritated eyes

**COMPOSITION** • 0.1% Euphrasia fluid extract, 0.1% Chamomile fluid extract, Disodium edetate, Polyhexaméthylène biguanide (PHMB). Isotonic solution buffered to pH 7.2. Does not contain: Chlorhexidine, Thimerosal, Benzalkonium Chloride, Phenoxyethanol and components from animal origin.

**INDICATIONS** Relieves allergic conjunctivitis and hay fever • Relieves eyelids and periocular area • Relieves, lubricates and protects sensitive eyes • Soothes, moisturises and refreshes red and irritated eyes. • Ideal for itching, burning, foreign body sensation. • Ideal in case of non-traumatic / non-pathological alterations of the tear film. • Provides relief from eye stress due to: excessive exposure to sunlight, wind, dust, chlorine, air conditioning and pollution, fatigue and irritation, prolonged use of screens and contact lenses. • Suitable for contact lens wearers.

**INSTRUCTIONS FOR USE** • The product can be used with the eye open or closed. • Shake before use. • Hold the bottle firmly, place the thumb on the dispenser taking care not to cover the nebuliser with your finger. • Hold the nebuliser about 10 cm from the eye, positioned to the side. • If you use the product with your eyes open, keep looking forward so that the bottle is at the lateral edge of the visual field. • Apply pressure to the dispenser to produce an effective mist. • It is advisable to spray twice for each application, repeat one or more times a day. • Dispose of the device according to local laws.

**WARNINGS** • In case of eye irritation, stop using the product and contact a clinician. • Do not use in case of allergy or hypersensitivity to any of the product's component. • Keep away from heat sources, store in a dry place at temperatures between 5 and 35 °C. • Keep out of the reach of children. • Do not use the product if the bottle appears damaged. • Do not use the product after the expiry date. • Use within 90 days of first opening. • Sterile until opened. • Do not ingest. • Do not let the tip of the bottle touch your eye or eyelid. • Close the bottle tightly after each use.

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